



## **ProTech™ BOIL-OUT COMPOUND**

Assures thorough wax elimination from:

FULL DENTURE MOLDS  
PARTIAL DENTURE MOLDS  
CROWN AND BRIDGE MOLDS  
ETC.

Keeps the boil-out tank clean and free from wax build-up.

### **DIRECTIONS**

Mix one (1) heaping tablespoon of Boil-Out Compound per quart of boiling water. Boil-out wax as usual. Rinse thoroughly with clean, hot water to remove any remaining film.

### **WARNING**

ProTech™ Boil-Out Compound is NOT RECOMMENDED for ALUMINUM FLASKS. Due to the chemical properties of the compound, aluminum products will be covered with a black coating of aluminum oxide, which can be removed by polishing.